


The Single-Solution Balloon Catheter for All Your Stent Graft Procedure Needs

Versatile

- For use in thoracic and abdominal endograft procedures
- Widest range of inflation diameters (10 mm – 46 mm)¹
- Lowest crossing profile stent graft modeling balloon¹
- Compatible with 12F sheath

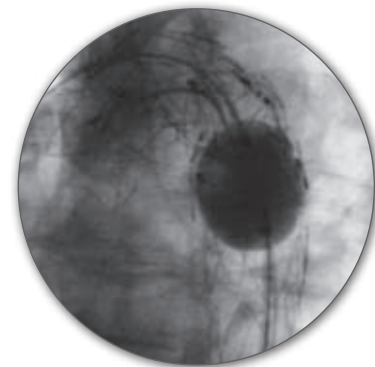


Reliable

- Consistent inflation and deflation time
- Stable expansion with minimum balloon overhang to reduce risk of vessel trauma
- Dependable expansion even after multiple inflations and deflations²



Conformable

- Improves conformability to vessel wall
- Expands kinks and smoothes creases from the graft material
- Compliant balloon expansion

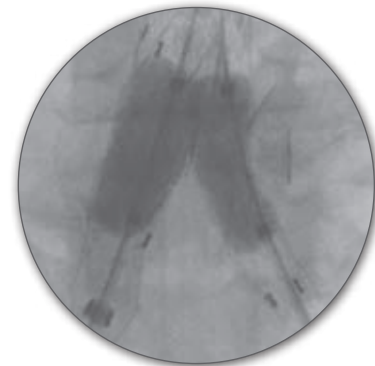


Reliant®

STENT GRAFT BALLOON CATHETER

Multiple Purposes, Single Solution

Clinical Uses Include:

- Abdominal and thoracic use
- Endograft modeling
- Endoleak sealing support

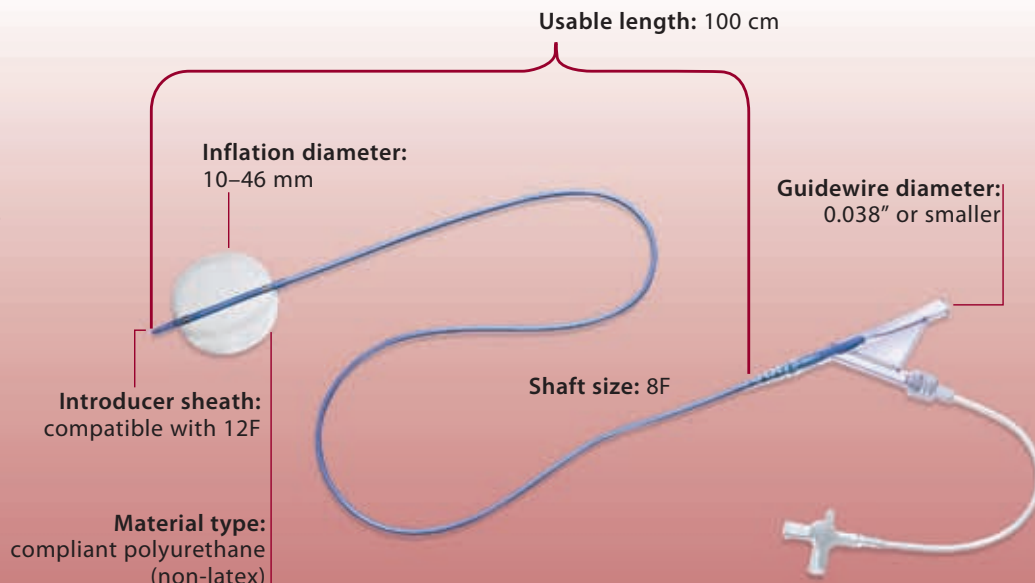

Widest Range of Balloon Inflation Diameters

Balloon Inflation Table

46 mm Balloon	
Diameter (mm)	ml (cc)
10	3
20	9
30	19
40	41
46 ³	60

CAUTION: This table is only a guide. Balloon expansion should be carefully monitored under fluoroscopy. Do not exceed maximum inflation diameter (46 mm). Rupture of balloon may occur.

Product Information

Reliant Stent Graft Balloon Catheter⁴

Product Code	Inflation Diameter (mm)	Shaft Size (F)	Usable Length (cm)	Sheath Compatibility (F)
AB46	10–46	8	100	12

Please reference appropriate product *Instructions for Use* for a more detailed list of indications, warnings, precautions and potential adverse events.

Notes:

1. Reliant® Stent Graft Balloon Catheter *Instructions For Use*, Cook Coda® Balloon Catheter *Instructions For Use*, Gore Tri-Lobe *Instructions for Use*.
2. Data on file Medtronic, Inc. Maximum of 20 inflations, deflations.
3. Maximum inflation diameter.
4. Does not contain latex.

www.medtronic.com

Medtronic Vascular

3576 Unocal Place
Santa Rosa, CA 95403
USA
Telephone: +1.707.525.0111

Medtronic BV

Earl Bakkenstraat 10
6422 PJ Heerlen
The Netherlands
Telephone: +31.45.566.8000
Fax: +31.45.566.8668

Medtronic Australasia

4/446 Victoria Road
Gladesville, NSW 2111
Australia
Telephone: +61.2.9879.5999

Medtronic Latin American Operations

3850 Victoria St. N. MS-V310
Shore View, MN 55126
USA
Telephone: +1.763.514.8083

Medtronic Europe SA

Route du Molliau
CH-1131 Tolochenaz
Switzerland
Telephone: +41.21.802.7000

Medtronic International LTD

Suite 1602 16/F, Manulife Plaza
The Lee Gardens, 33 Hysan Avenue
Causeway Bay, Hong Kong
Telephone: +852.2891.4456

Medtronic of Canada LTD

6733 Kitimat Road
Mississauga, ON L5N 1W3
Canada
Telephone: +1.905.826.6020